

GLOBADENT[®] RTU

Aldehyde-free agent for disinfecting dental impressions and dentures

ready-to-use solution

Product properties

- amine-based ready-to-use solution
- for washing and disinfecting dental impressions, dentures, dental bridges, dental crowns, and dental spoons.
- used for cleaning and disinfecting silicone, alginate, polyether rubber, polysulphide and hydrocolloid impressions, prosthetic works, etc.
- quick exposure time (5 min)
- plaster cast model accuracy thanks to exceptional cleaning power (blood, saliva, etc)
- broad spectrum of efficacy (Tbc, HIV, HBV, etc)

Ingredients

Amines, quaternary ammonium compounds, polyglycol ether, guanidine, corrosion inhibitor

Spectrum of efficacy

The product has bactericidal (incl. tuberculocidal), virucidal (incl. HBV, HCV, HIV), and fungicidal efficacy.

Usage

After taking a dental impression or trying the dentures, immerse them for 5 minutes in a container closed with a lid and filled with the working solution. Make sure that all surfaces that are disinfected are covered with the solution. After the exposure period has elapsed wash the impressions/dentures with water (at least drinking water quality). The solution may be used for up to seven working days, depending on the work load.

Note! It is not recommended to keep the dental impressions/dentures etc in the solution beyond the exposure time.

Note! Avoid contact with glutaraldehyde!

Warning

Harmful if swallowed. Causes skin irritation. Causes serious eye irritation. Very toxic to aquatic life. Avoid release to the environment. Wear protective gloves/eye protection/face protection.

Physical-chemical properties

Physical appearance: *colourless fluid*
 Scent: *distinctive to amine*
 pH: *9–10 (20°C)*
 Solubility: *completely soluble in water*
 Density: *1,01 g/cm³ (20°C)*

Storage

See the best before date on the package. Store at 0–30°C with the cap tightly closed, keep away from direct sunlight.

Utilization

Unused product residues and empty packages should be given to licensed waste management.

Compliance to standards

EN 1276, EN 13727, EN 14562, EN 14348, EN 14476+A1, RKI/DVV, DGHM

Package

5L container



Goodpoint Chemicals OÜ

<http://www.goodpointchemicals.com>

Urda tee 3, Jälgimäe, Saku parish, Estonia

Contact +372 6626 511 | Fax: +372 6626 522

info@goodpointchemicals.com