

GLOBAQUAT[®] WIPES med

Material-friendly alcohol-free wet cleaning and disinfectant wipes

Product properties

- odorant- and colorant-free
- quick exposure time (1 min)
- suitable for various kind of materials
- alcohol- and aldehyde-free
- excellent cleaning performance
- convenient to use

Ingredients

Three different quaternary ammonium compounds
0,75%

Spectrum of efficacy

Globaquat[®] Wipes are bactericidal, virucidal (HBV, HCV, HIV, Rota, Vaccinia, Influenza etc) and fungicidal.

Exposure time

1 minute even in dirty conditions.

Usage

Pull the wipes out until perforated spot according to the amount necessary and if needed support with other hand in order to tear away. All surfaces should be wiped with Globaquat[®] Wipes med, make sure that the surface is covered with disinfectant. Throw the used wipe away. In order to avoid the drying of wipes, close the flip.

Range of use

Used for cleaning and disinfecting medical devices (ECG, ultrasound probes, infant incubators), dental patient chairs, acrylic surfaces, different types of plastic, wires, mattresses etc.

Physical-chemical properties

Physical appearance: *damp cotton wipes, that are permeated with transparent, slightly opalescent and slightly viscose liquid*

Scent: *distinctive*

pH: *7,0-7,2*

Density: *0,98 – 1,01 g/cm³ (20°C)*

Storage

See the best before date on the package. Store at room temperature, keep away from direct sunlight and avoid freezing to assure the quality of the product.

Utilization

Unused product residues and empty packages should be given to licensed waste management.

Compliance to standards

EN 14476, EN 1276, EN 1040, EN 13697, EN 1275

Other test methods

DGHM, DVG, DVV/RKI

„Effect on Acrylic Glass“ Röhm GmbH

Package

120 wipes in a can, dimensions of a wipe 20 x 20cm
450 wipes in a bucket, dimesions of a wipe 18 x 20cm



Goodpoint Chemicals OÜ

<http://www.goodpointchemicals.com>

Urda tee 3, Jälgimäe, Saku parish, Estonia

Contact +372 6626 511 | Fax: +372 6626 522

info@goodpointchemicals.com